




Patch Tips


CeQur Simplicity™
MEALTIME INSULIN SIMPLIFIED™

PATCH SITES

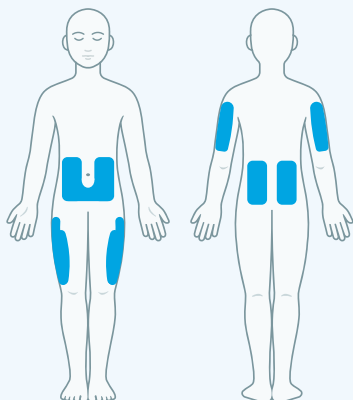


Place Patch in any blue areas.
Ensure you can click both buttons.

PATCH PLACEMENT

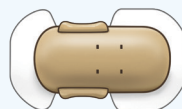

Place horizontally
on abdomen
& backside

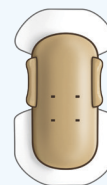

Place vertically
on arms
& thighs

- Remove excess body hair where Patch will be placed
- Avoid lotion, sunscreen, perfume or medication around Patch area
- Use 2 alcohol wipes to clean skin: 1st to lift dirt/oils from skin, 2nd to remove dirt/oil from skin
- Allow area to fully dry prior to placing Patch
- Avoid wetting Patch for 3 hours after placement
- Rotate sites with each Patch change

If you need additional help **sticking the Patch securely and removing it gently**, below is a list of products that may help.

HELP YOUR PATCH STICK



Product*	Description
Skin Tac™ torbot.com	A hypo-allergenic and latex-free “tacky” skin barrier in a wipe and liquid form
SKINPREP® Protective Wipes smith-nephew.com	Protective wipes forms a barrier that allows skin to breathe so tapes and films adhere better
IV Prep™ Wipes smith-nephew.com	Leaves a sticky film which may help reduce skin trauma
AllKare® Wipes convatec.ca	Latex-free barrier wipes provides a film layer on skin to help protect against irritation
Mastisol® Adhesive eloquesthealthcare.com	Non-irritating, non-water soluble liquid adhesive

HELP REMOVE YOUR PATCH



Product*	Description
Baby Oil (generic) by different manufacturers	Soft moisturizer
TacAway® Adhesive Remover torbot.com	Removes adhesive residue on skin
UNI-SOLVE™ Wipes smith-nephew.com	Softens and removes residue from your skin



Mild, temporary bleeding & bruising at application sites may happen. If you have active bleeding at the application site, remove the Patch and replace with a new Patch in a new location. Apply firm pressure to the bleeding area with a sterile or clean cloth. **Any questions, call/text CeQur Cares at 1-888-552-3787.**